

# OpClear<sup>®</sup>

## Quick Guide

φ	∟	Length	Model
10mm	0°	300mm	CS-10-00-300
10mm	0°	315mm	CS-10-00-315
10mm	0°	330mm	CS-10-00-330
10mm	30° / 45°	330mm	CS-10-30-330
10mm	30° / 45°	315mm	CS-10-30-315
10mm	30° / 45°	300mm	CS-10-30-300
5mm	0°	290mm	CS-05-00-290
5mm	0°	315mm	CS-05-00-315
5mm	0°	330mm	CS-05-00-300
5mm	30° / 45°	330mm	CS-05-30-300
5mm	30° / 45°	315mm	CS-05-30-315
5mm	30° / 45°	290mm	CS-05-30-290



5 060346 800547

CS-CUOM-M5

[www.ciphersurgical.com](http://www.ciphersurgical.com)



**CONTINUOUS VISION**  
A NEW STANDARD OF CARE

CLARITY CONTROL CONTINUITY

# OpClear® Quick Guide

CIPHER  
SURGICAL

**BEFORE**

1. Plug the power cord into a standard wall outlet.
2. Connect the CO<sub>2</sub> gas line to the CO<sub>2</sub> inlet on the back of the unit.
3. Connect the CO<sub>2</sub> gas cylinder to the CO<sub>2</sub> inlet.
4. Turn the power switch on the front panel to the 'I' (On) position.
5. Press the power button on the front panel. The display shows 'Adu' and 'PEd'.
6. Select the 'Auto' mode using the arrow keys on the control panel.
7. Select the '7' setting on the control panel.

**DURING**

1. The display shows 'Auto' and 'PEd'.
2. The CO<sub>2</sub> gas is delivered to the patient through the end effector.

**AFTER**

1. Disconnect the end effector from the patient.
2. Turn the power switch on the front panel to the 'O' (Off) position.
3. Disconnect the CO<sub>2</sub> gas line from the back of the unit.
4. Turn the power switch on the front panel to the 'I' (On) position.
5. Disconnect the power cord from the wall outlet.

CLARITY CONTROL CONTINUITY